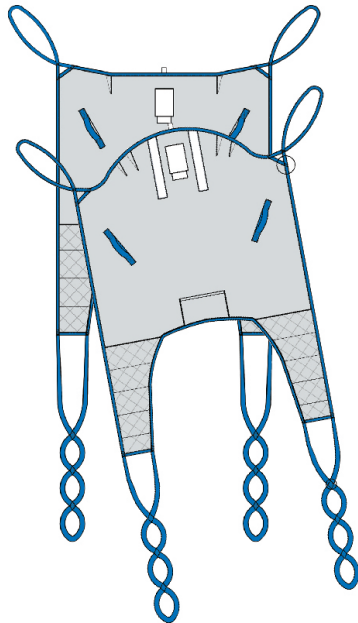
dutchblue.com

SLINGS

GENERAL PURPOSE SLING LOOP/CLIP BATH SLING LOOP/CLIP

User Manual | Gebrauchsanweisung | Mode d'emploi
Gebruikershandleiding | Ръководство за употреба

REF 40014, 40015, 40016, 40017, 40018, 40020, 40021, 40022,
40023, 40024, 41014, 41015, 41016, 41017, 41018, 41020,
41021, 41022, 41023, 41024

dutchblue.com


SLINGS

English	4
General purpose sling Loop	
General purpose sling Clip	
Deutsch	10
Transfergurt Schlaufen	
Transfergurt Clips	
Français	22
Harnais pour amputé à boucles	
Harnais à usage général à clips	
Nederlands	22
Standaard tilband met lusbevestiging	
Standaard tilband met clipbevestiging	
Български	28
Универсални сапани с примки	
Универсални сапани с клипс	

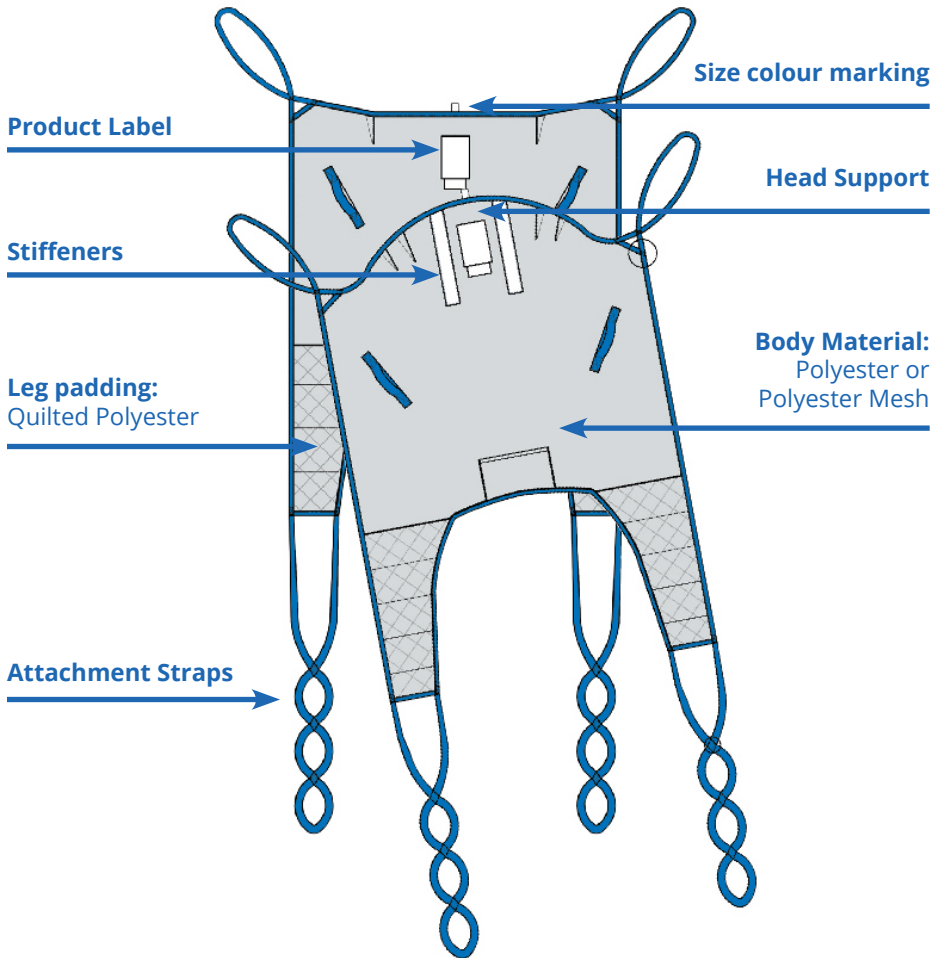ENGLISH

Warning!

- Always check the label is readable and that the sling is in good condition without signs of wear and tear.
- Always make sure not to exceed the Safe Working Load (SWL) defined for the product.
- Always read the instruction manual for the lifting equipment to be used with the sling.

Technical Specifications

REF	Name	SWL	Material	Sizes
40014-40024	General purpose sling Loop	Max: 275kg /605 lbs.	Polyester	S-XXL
41014-41024	General purpose sling Cip	Max: 275kg /605 lbs.	Polyester	S-XXL

Washing Instructions

Intended Use

The Universal Sling is designed with split legs and shaped body support for users with good head control.

Suitable for most transfers in a seating or laying position for users with good head control.

The Universal Sling HS (with Head Support) is designed with split legs and full shaped body for users who requires head support. Suitable for most transfers in a seating or laying position and where user have little neck and/or head control.

Loop Straps version is for use with lifters/hoists and hanger bars with loop attachment hooks.

Keyhole clip version is for use with lifters/hoists and hanger bars with 4-point stud fixings.

The Universal Sling will assist the caregiver for lifting and transfer a patient in either a sitting or laying position, from and to the bed and/or wheelchair.

Before and after every use

1. Always perform a risk assessment before starting the specific lifting session in order to make sure the product model is correct and that the lifting procedure is fully understood.
2. Before using the sling, read this instruction manual and the instruction manual for the lifting equipment to be used.
3. When soiled or stained the sling must be washed before use.
4. **Warning:** Check the sling for any fraying, loose stitching, attachments or any deteriorations.
5. After use, check sling according to the step 3 and 4 above and store in a dry area.

Attachments

The Universal Loop Sling will fit most lifts with standard hangerbars with loop attachment hooks.

The Universal Keyhole Clip Sling will fit most lifters with 4-point hanger bars with clip attachment studs.

Always check lifter instruction manual for details.

Service & Maintenance

The sling shall be checked by authorised personnel every 6 months according to ISO10535.

Customer support/Trouble shooting

- Please refer to the manual for the lifting equipment used.
- In doubt please contact your dealer who supplied the product.

Expected service life & End of Life Disposal

- Expected Service Life varies between 1–5 years, depending on usage and number of washing cycles. When washed once a week the sling has an Expected Service Life of 2 years. Expected Service Life is based upon a correct usage and washings procedure in accordance with this IFU. The sling are washable according to the marking on the label.
- Slings including stiffeners/stabilizers, padding material, any other textiles or polymers or plastic materials etc. should be sorted as combustible waste.
- The packaging bag consist of soft plastic and should be recycled according to local legislations.

Storing and Shelf life

The sling should be stored in a dry and clean environment. The shelf life is 3 years in original packaging. Store in temperatures between 0°-40° C (+32°-104°F).

Safe Working Load (SWL)

Max 275kg / 605 lbs.

Product care & Maintenance

- Always follow the instructions on the product label and check the sling for deteriorations.
- Mark the sling with the specific patient name and keep in dry and clean environment.

Applying the Universal Sling in Chair

1. Lean the user slightly forward and position the Sling behind the user's back and head. The label side facing outward and the center line of the sling aligned with the user's spine. Place the hand in the position pocket and carefully push the sling as far down the user's lowerback as possible. (See picture 1)

Place the sling legs alongside the user's legs. Fold the sling leg (See picture 2) and carefully feed it underneath the user's legs.

After positioning the sling legs (see pic 3,4) cross the leg straps by putting one through the other. (See picture 5)

Recline the user into the sling, ensuring the arms are contained within the sling.

Note! If required, two flat slide sheets can be used to apply the sling. (See pictures 6,7,8,9)

2. Position the hanger bar carefully above the user. (See picture 10)
3. Place the four loop straps on the hanger bar attachment hooks and pull the straps down until locked in position. (See picture 11,12)
4. Check user position and carefully make the required adjustments to achieve the best comfort for the user. Check the support of the head and that arms and legs are in the right positions. (See picture 13)
5. Raise the hanger bar until all straps start to tighten and become tense enabling you to check ALL the straps are securely attached. (See picture 14,15)

1

2

3

4

5

6

7

8

9

10

11

12

13

14

6. Carefully perform the lift, whilst remaining vigilant of the user's position and comfort. (See picture 16)
7. Once the process of transfer is complete, return the user to a chair or bed and slowly lower the hanger bar until all straps are loose. (See picture 17)
8. Release the loop straps from the hanger bar by pulling the strap upwards. (See pic 18,19)

Note: Always check the lifter instruction manual for details

15

16

17

18

19

Applying the Universal Sling in Laying position

1. Roll the user to the side, facing away from you. Fit the sling by folding it on the middle. The label side facing outwards. Then place the sling lengthwise with the users back. (See picture 1)

1

2

Roll up top half of the sling and gently tuck it under the user. (See picture 2,3,4)

2. Roll user to the other side, now facing you. Gently pull the top half of the sling through, when fully unraveled flatten it. (See picture 5)

3

4

Roll the user to a backside position. (See picture 6)

Note! If required, two flat slide sheets can be used to apply the sling. (See picture 7,8).

3. Bend the users leg closest to you and insert sling leg under the users thigh. (See picture 9,10)

5

6

7

8

Unbend the knee and redo procedure on the users opposite leg. (See picture 11)

After positioning the sling legs properly, cross the leg straps by putting one through the other. (See picture 12)

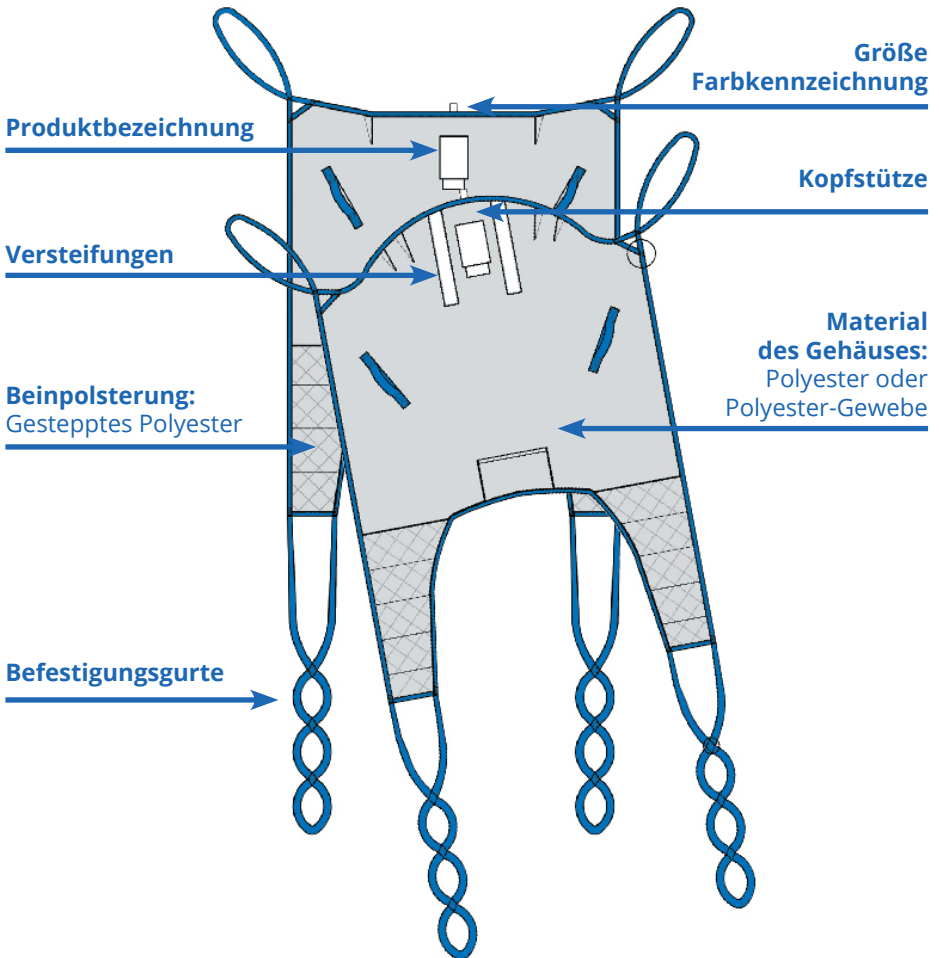
4. Position the hanger bar carefully above the user. (See picture 13)
5. Place the four loops straps on the hanger bar attachment hooks and pull the straps down until locked in position. (See picture 14,15)
6. Check user position and carefully make the required adjustments to achieve the best comfort for the user. Check the support of the head and that arms and legs are in the right positions. (See picture 16)
7. Raise the hanger bar until all straps start to tighten and become tense, enabling you to check that ALL the straps are securely attached. (See picture 17)
8. Carefully perform the lift, whilst remaining vigilant of the user's position and comfort. Remember to support the users head when raising from the bed. (See picture 18,19)
9. Once the process of transfer is complete, return the user to a chair or bed and slowly lower the hanger bar until all straps are loose. (See picture 20)
10. Release the loop strap from the hanger bar by pulling the strap upwards. (See picture 21,22)

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Scheitlin-Papier AG